

These respirators are approved only in the following configurations:

Respirator Components			
TC-	Protection ¹	DC365N95HC	Cautions and Limitations ²
84A-9250	Surgical/N95	X	ABCJM NOS

1. PROTECTION

N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. CAUTIONS AND LIMITATIONS

- A. Not for use in atmospheres containing less than 19.5% oxygen.
- B. Not for use in atmospheres immediately dangerous to life or health.
- C. Do not exceed maximum use concentrations, established by regulatory standards.
- J. Failure to properly use and maintain this product could result in injury or death.
- M. All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N. Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- Q. Refer to User's Instructions, and / or maintenance manuals for information on use and maintenance of these respirators.
- S. Special or critical User Instructions and/or specific use limitations apply. Refer to User Instructions before donning.



DONNING

Follow these instructions each time the respirator is worn.

1. Cup the respirator in the hand, allowing the head straps to hang freely.
2. Hold the respirator under the chin with the nosepiece facing outwards.
3. Pull the lower strap up over your head and place around the neck, below the ears.
4. Holding the respirator against your face with one hand, grasp the top head strap with other hand.
5. Place the top head strap above your ears, around the crown of your head.
6. Place hands on each side of the respirator and move the respirator slightly right, left, up and down, to adjust the position of the respirator and achieve the most optimum fit to your face.
7. Then place both index fingers on the nose clip and apply pressure onto the nose clip, forming it to your nose to get the best fit possible on your face.

USER SEAL CHECK

1. Perform a user seal check before each use.
2. Place both hands over the respirator-do not disturb the position of the respirator.
3. Exhale sharply. A positive pressure should be felt inside the respirator.
4. If you detect air leaking in when you inhale, readjust the respirator by:
 - a. applying additional pressure on the nose clip to assure a secure seal on the nose;
 - b. adjusting the position of the respirator on your face; and/or
 - c. adjusting the position of the head straps.

WARNING: If you cannot achieve satisfactory fit, do NOT enter the contaminated area. See your Supervisor.



DOFFING

1. Without touching the respirator and handling it only by the straps, gently lift the bottom strap from around your neck and pull it up and over your head.
2. Then lift the top strap and pull the respirator away.
3. Dispose of the used respirator in accordance with applicable local regulations.
4. Perform hand hygiene.

